

## **Actimove® Ankle Support for Kids** **INSTRUCTIONS FOR USE**

### **Products:**

- **Actimove® Ankle Support for Kids: small child**
- **Actimove® Ankle Support for Kids: youth**

**Ankle support for kids, designed for post-sport injury recovery or prevention of re-injury. It can be used for reducing swelling, preventing ankle injuries, addressing mild strains, and providing support in cases of ankle instability and weakened muscles.**

**Medical device. For information on the risks, read the instructions for use below or ask your healthcare professional!**

**Actimove® products are available from pharmacies or medical supply stores. To purchase the product medical prescription is not required.**

### **Instructions for use:**

Open the box and remove the product.

Place the child's foot into it so that the Actimove® logo is on the upper part of the device towards the foot. If the device is properly positioned the heel remains unrestricted.

The recommended period for wear varies but is usually for the duration of exercise or physical activity, i.e. from 30 to 180 minutes. In acute conditions, the ankle support can be worn all day.

Wearing the product at night and during rest periods is not recommended as excessive use may cause muscle weakness, the stabilising and retaining functions of the muscle may be impaired.

When worn for therapeutic purposes, please discuss the necessary usage period of the device with a healthcare professional.

The product packaging shall be disposed in the appropriate separate waste bin.

Washable at 40 °C.



### **Contraindications:**

- Diseases associated with peripheral venous occlusion (phlebo-thrombosis, thrombo-phlebitis)
- Peripheral arterial circulatory disorders
- Venous inflammation caused by microorganisms (septic phlebitis)
- Skin diseases (dermatoses)
- Open wounds, any surface lesions with disruption of skin continuity, ulcers
- Advanced peripheral neuropathy due to nerve damage or degeneration in the extremities
- Intolerance to the material of the product

**Manufacturer:** BSN Medical GmbH, Germany, Schützenstrasse 1-3 22761 Hamburg

**Distributor (seller):** Essity Hungary Ltd. Budakeszi Road 51.1021 Budapest. Any unexpected occurrences related to the use of this device please report to BSN Medical GmbH BSN Medical GmbH, Schützenstrasse 1-3 22761 Hamburg, Germany or Essity Hungary Ltd, (Budakeszi Road 51. 1021 Budapest) or to the local competent authority (National Institute of Pharmacy and Food (OGYÉI), 1051 Budapest, Zrínyi str. 3, Tel.: (1) 8869-300, E-mail: amd.vig@ogyei.gov.hu, 1372 P.O. Box 450