

INSTRUCTIONS FOR USE

Actimove Knee Brace Wrap Around Simple Hinges Condyle Pads

The product is designed to provide knee stability.

Areas of application:

- ACL, LCL, MCL, PCL sprains and tears
- Medial/lateral meniscus tears
- Patellar instability
- Patellar tendonitis
- Patellar cartilage tumour
- Osgood-Schlatter disease


Contraindications:

- Diseases associated with peripheral venous occlusion (phlebo-thrombosis, thrombo-phlebitis)
- Peripheral arterial circulatory disorders
- Venous inflammation caused by microorganisms (septic phlebitis)
- Skin diseases (dermatoses)
- Open wounds, any surface lesions with disruption of skin continuity, ulcers
- Advanced peripheral neuropathy due to nerve damage or degeneration in the extremities
- Intolerance to the material of the product

Features:


- A device that provides proper medical compression and even therapeutic heat, thereby facilitating pain relief and promoting recovery.
- Made with COOLMAX® AIR TECHNOLOGY (Neoprene-free), its fast drying perforated material allows excellent ventilation, providing the retention of skin's breathability and thereby offering above-average comfort.
- It relieves knee pains resulting from soft tissue and ligament injuries, as well as bone-joint inflammations, through the strong stabilizing effect provided by the medial and lateral (inner-outer side) straps.
- Thanks to its front fastening system, the device provides painless, comfortable, and irritation-free use during both putting on and taking off.
- The elastic buckle and loop holder allow for a better fit, while its hook-and-loop fastening system enables safe fixation.
- The Actimove® Knee Brace can be used on both the left or the right knee.

Application:

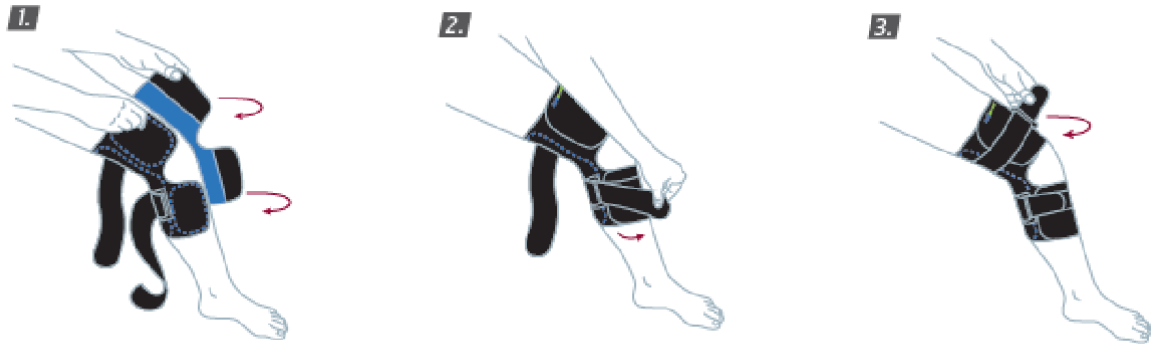

The period for wear varies but is usually for the duration of exercise or physical activity, i.e. from 30 to 180 minutes. In acute conditions, the knee brace can be worn all day.

Sizing chart:

	SIZE	XS	S	M	L	XL	XXL	XXXL
		30.5 - 35.5 cm 12" - 14"	35.5 - 40.5 cm 14" - 16"	40.5 - 46 cm 16" - 18"	46 - 51 cm 18" - 20"	51 - 56 cm 20" - 22"	56 - 61 cm 22" - 24"	61 - 66 cm 24" - 26"

Ingredients:

Elastane, polyamide, polyester, polyoxymethylene, PVC, SBR (Styrene Butadiene Rubber) stainless steel.

Neither the product nor the packaging contains natural rubber latex or phthalates.

Storage:

In a dry, clean place, protected from direct sunlight.

Cleaning:

Maximum washing temperature: 40°C. Bleach usage is prohibited. Lay flat to dry. Do not iron. Dry cleaning is not allowed.



Expiration date:

The products have a shelf life of 5 years.

See the expiration date on the packaging.

Manufacturer: BSN Medical GmbH, Schützenstrasse 1-3 22761 Hamburg, Germany.

Distributed by Essity Hungary Ltd. Budakeszi Road 51. 1021 Budapest

Note

Please immediately report any serious and unexpected events related to the use of the device to BSN medical GmbH (BSN Medical GmbH, Germany, Schützenstrasse 1-3, 22761 Hamburg) or Essity Hungary Ltd. (Budakeszi Road 51. 1021 Budapest), as well as to the relevant local authorities (National Institute of Pharmacy and Nutrition - OGYÉI, 1051 Budapest 3 Zrínyi str. Budapest, Tel: (1) 8869-300, E-mail: amd.vig@ogyei.gov.hu, 1372 P.O. Box 450).



Closing date: July 22, 2022.