

Actimove® Wrist Stabilizer for Kids
INSTRUCTIONS FOR USE

Wrist stabilizer for tendon and joint inflammation, strains, and minor injuries, as well as for postoperative use and after cast removal.

Medical device. For information on the risks, read the instructions for use below or ask your healthcare professional!

Actimove® products are available from pharmacies or medical supply stores. To purchase the product medical prescription is not required.

Instructions for use:

Open the box and remove the product.

Open the hook-and-loop fasteners and place the metal splint according to the lateral side (for right-hand use, put it into the left-side, longitudinal small pocket; for left-hand use, put it into the right side, so that the curved metal plate follows the arc of the palm rest during use, with its bent part facing upward and inward).

Slide the product onto the child's hand in a way that the palm rest follows the arc of the supporting metal splint and lies along it. For proper stability, adjust the necessary tightness using the three hook-and-loop straps that go around the forearm. Secure the upper, narrower strap by pulling it through between the thumb and forefinger on the palm side of the product.

The recommended period for wear varies but is usually for the duration of exercise or physical activity, i.e. from 30 to 180 minutes. In acute conditions, the wrist stabilizer can be worn all day.

Wearing the product at night and during rest periods is not recommended as excessive use may cause muscle weakness, the stabilising and retaining functions of the muscle may be impaired.

When worn for therapeutic purposes, please discuss the necessary usage period of the device with a healthcare professional.

The product packaging shall be disposed in the appropriate separate waste bin.

Washable at 40 °C.



Contraindications:

- Diseases associated with peripheral venous occlusion (phlebo-thrombosis, thrombo-phlebitis)
- Peripheral arterial circulatory disorders
- Venous inflammation caused by microorganisms (septic phlebitis)
- Skin diseases (dermatoses)
- Open wounds, any surface lesions with disruption of skin continuity, ulcers
- Advanced peripheral neuropathy due to nerve damage or degeneration in the extremities
- Intolerance to the material of the product

Manufacturer: BSN Medical GmbH, Germany, Schützenstrasse 1-3 22761 Hamburg

Distributor (seller): Essity Hungary Ltd. Budakeszi Road 51.1021 Budapest. Any unexpected occurrences related to the use of this device please report to BSN Medical GmbH BSN Medical GmbH, Schützenstrasse 1-3 22761 Hamburg, Germany or Essity Hungary Ltd, (Budakeszi Road 51. 1021 Budapest) or to the local competent authority (National Institute of Pharmacy and Food (OGYÉI), 1051 Budapest, Zrínyi str. 3, Tel.: (1) 8869-300, E-mail: amd.vig@ogyei.gov.hu, 1372 P.O. Box 450